MOBI-ROLL

Knie Roller

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Instruction manual

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Istruzioni d'uso

Mobi-Roll (MR-22)

Mobi-Roll S (MRS-23)

Mobi-Roll B (MRB-23)

Mobi-Roll CC (MRCC-23)

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In English: READ THE INSTRUCTIONS BEFORE USE

Contents

- ✓ Introduction;
- ✓ Indication;
- ✓ Product description;
- ✓ Description of purpose;
- ✓ Contra-indications;
- ✓ Care and maintenance;
- ✓ Warnings and safety instructions;
- ✓ Assembly instructions;
- ✓ Operating instructions;
- ✓ Product parameters.

Introduction

Please read this user manual carefully before use.

Before using the Mobi-Roll, you should be trained by your doctor or physiotherapist in the correct use of the Mobi Roll, including adjustment to the appropriate height.

Indication

The Mobi-Roll is designed for people who have problems with balance, weakness in standing, require complete relief of one foot or need a fixed immobile support to help them walk.

Product description

Specially designed wheels/tyres enable a full mobility experience thanks to the suspension. The rear wheel suspension effectively reduces shocks and protects the injured foot. The brakes have a parking function that can be operated by either of the two handbrake levers. Both brake levers act on both rear wheels. The parking function provides a stable stand if required. Sturdy handles provide a good grip, even in wet weather. The steering not only ensures improved stability, but with a turning radius of just 1.40 m, it also fulfils all the requirements for easy handling around the house. The weatherproof, lockable and removable bag provides support for all types of indoor and outdoor transport.

The handlebar and saddle height can be easily adjusted many times without tools. The saddle provides comfortable support for your damaged leg with an extra moulded recess for the knee. A bell provides additional protection when travelling. In addition to the compact folding function and easy stowage of the Mobi-Roll, e.g. in the boot of a car, the low weight of 11 kg ensures easy handling.

Intended use

The Mobi-Roll is designed to compensate for chronic or temporary functional limitations in the foot, ankle and/or distal lower leg following trauma or surgery. It facilitates locomotion for people with balance problems or weaknesses when standing.

Contraindications

Do NOT use the device if you are in any of the following situations:

- ✓ Visual impairments;
- ✓ Severe balance problems;
- ✓ Disabilities or injuries to both legs.

Care and maintenance

- ✓ Clean with a damp cloth and non-abrasive cleaning agent.
- ✓ Clean the wheels with warm water and a mild detergent. Dry with a clean cloth.
- ✓ Ensure that all attached brackets and seat levers are secure at all times.
- ✓ Check the Mobi-Roll regularly to ensure that the brakes, seat lever, steering and wheels are working properly and that all parts are secure.
- ✓ Replace damaged, damaged or worn items immediately.
- ✓ If the handle is loose, DO NOT USE. Replace it or contact your dealer immediately.
- ✓ Make sure that the brakes are working correctly. Over time, the brake cables can stretch with use, they can be adjusted by a qualified dealer. Failure to do so may result in injury to the user.


Warnings and safety instructions

- ✓ Suitable for patients aged 7 and over.
- ✓ Check the function and proper attachment of the parts before the first use and before each journey. Check that the brake brakes and that the wheels run smoothly and quietly.
- ✓ The first test drive should take place on a large, smooth road surface where you can practise undisturbed.
- ✓ Riding on too steep a gradient can lead to undesirably high speeds, which can lead to falls due to abrupt and uncontrolled manoeuvres.
- ✓ Smaller steps, lowered kerbs or ruts should never be approached parallel or at an angle. There is a risk that the front wheels could lock abruptly and cause a fall. It is recommended that you approach the aforementioned bumps at a 90° angle.
- ✓ Collisions of all kinds must be avoided.
- ✓ Wearing long socks/trousers is recommended as, in rare cases, direct skin contact in the area of the knee pad can lead to heavy sweating and skin irritation.
- ✓ The maximum permissible load weight (rider weight + payload) of 136 kg must not be exceeded, nor must the maximum weight (5 kg) of the payload. The loading of sharp objects is prohibited.
- ✓ The speed of mobility should always be adapted to the current state of injury in order to enable the best possible protection/regeneration.
- ✓ The Mobi-Roll must be adjusted to the user's height before use. In addition, care should be taken when wearing a rail to ensure that it and the foot do not come into contact with the rear tyres/wheels.
- ✓ Lock the parking brake when parking the Mobi-Roll.
- ✓ The assistance of a third party is recommended when carrying or lifting.
- ✓ If you have any problems or questions regarding the use of the Mobi-Roll, please contact the manufacturer or your specialist dealer immediately. If you have any questions in connection with your medical treatment and therapy, please contact your doctor.
- ✓ Adverse reactions or incidents involving the device shall be reported to the manufacturer and to the authority of the Member State in which the user and/or patient is established.
- ✓ Do not hang anything on the handles or the frame of the Mobi Roll at any time. This can cause the Mobi-Roll to tip over, resulting in personal injury or damage to the Mobi-Roll. Smaller objects can be transported in the Mobi-Roll basket.
- ✓ DO NOT try to reach objects while using the Mobi-Roll. Reaching for these objects will lead to a change in the weight distribution and therefore to a risk of tipping.
- ✓ Do NOT walk/ride backwards while using the Mobi-Roll.
- ✓ All wheels must be in contact with the ground at all times during use. This ensures that the Mobi-Roll is correctly balanced.
- ✓ Always keep both hands on the handles when using the Mobi-Roll.
- ✓ Only use accessories and spare parts that have been authorised by the seller.
- ✓ When turning the Mobi-Roll, be careful and only turn while moving slowly.
- ✓ The Mobi-Roll is intended for individual use only. DO NOT attempt to push the Mobi-Roll while someone is sitting on the seat. This can lead to serious injury and damage to the Mobi-Roll.

Assembly instructions

All components

- 1 Shopping bag
2. Front handlebars
3. Knee support
4. Main frame
5. Front wheel frame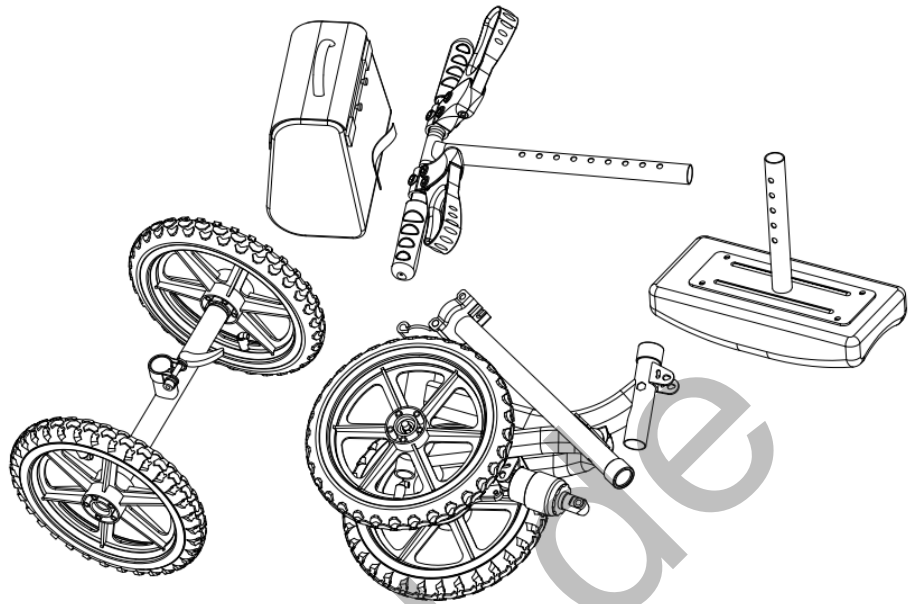


STEP 1: How to install the Mobi-Roll frame

Unpack the Mobi-Roll from the box.


Turn the outer tube of the handlebar to the vertical position and allow the locking pin to engage. You can help it to engage by pushing the locking pin to the side. Then lock the quick-release lever in the upper position and flip it over.

To reposition the handlebar, release the quick-release lever, fold it forwards and push it to the side to release the locking pin.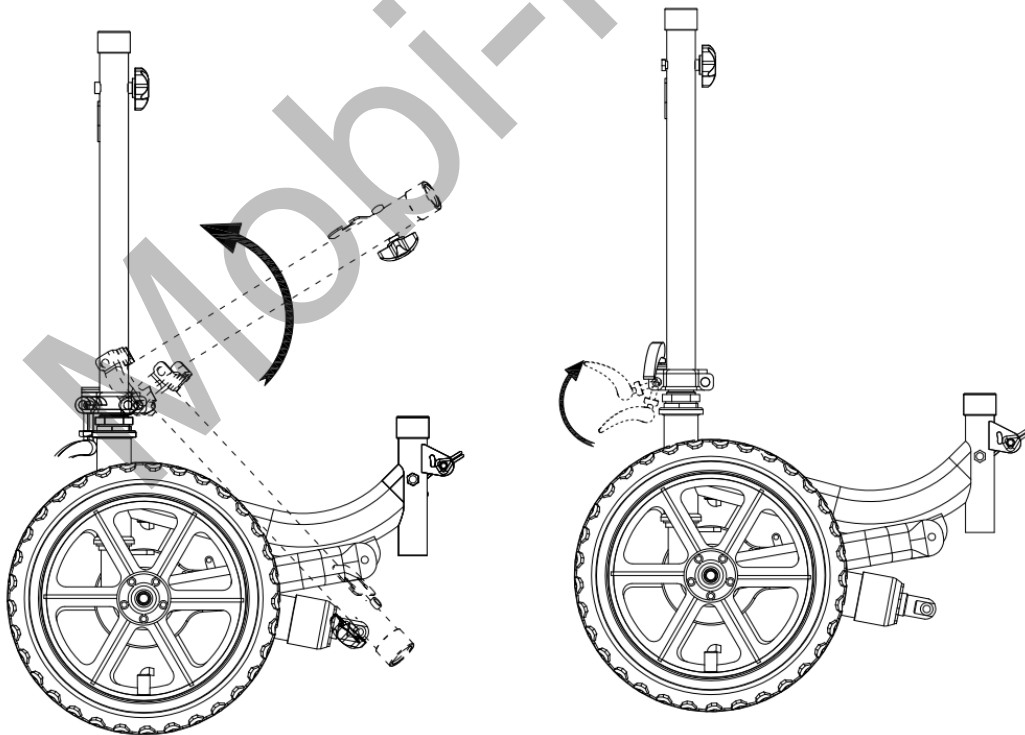

STEP 2: Fitting the rear frame


First fold the rear wheel carrier upwards until the upper hole of the shock absorber is aligned with the fastening hole. Insert the quick-release pin and fold down the bracket for fastening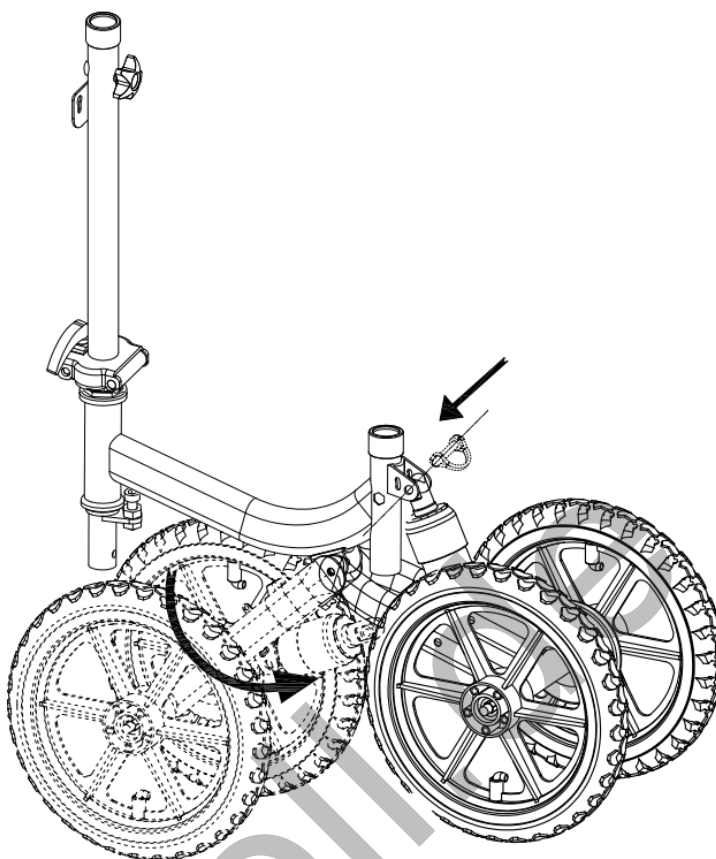

STEP 3: How to fit the front frame


Insert the frame into the front axle until the fastening hole is aligned; then insert the fastening screw with the hexagon head so that the hexagon head is countersunk halfway into the hexagon frame opening provided.

IMPORTANT: The screw head must be countersunk in the hexagonal opening of the frame when installed correctly.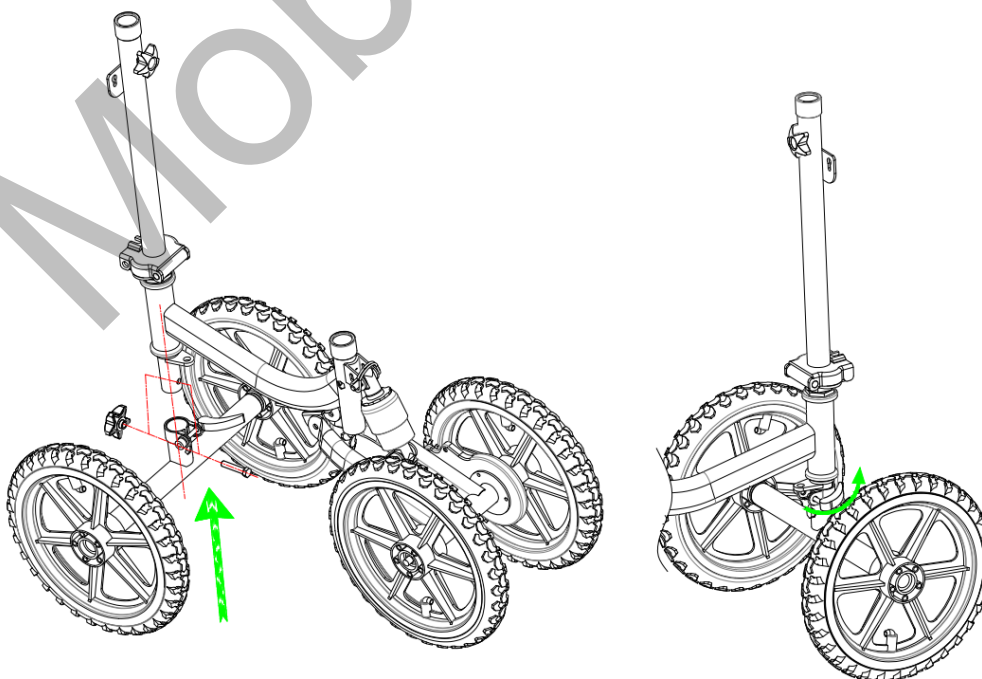

STEP 4: To fit the front handlebars

As shown: Insert the handrail frame into the frame as shown and insert the handrail frame hole. Align with the frame hole, thread the screw through the hexagonal end and ensure that the screw head is countersunk into the hexagonal frame opening.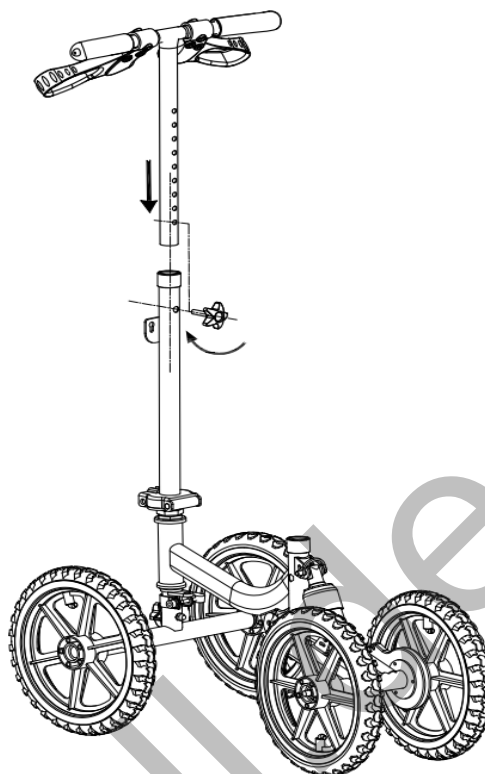

IMPORTANT: The screw head must be countersunk in the hexagonal opening of the frame when installed correctly.


STEP 5: Fit the knee pad

As shown: insert the tube of the knee pad into the frame until the correct height is reached and the mounting holes are aligned. Align with the frame hole, thread the screw through the hexagonal end and insert the screw head correctly in line with the handlebar assembly.

IMPORTANT: The screw head must be countersunk in the hexagonal opening of the frame when installed correctly.

STEP 6: Install the shopping basket

Place the two nails of the bag through the mounting hole of the frame at the same time and press them downwards

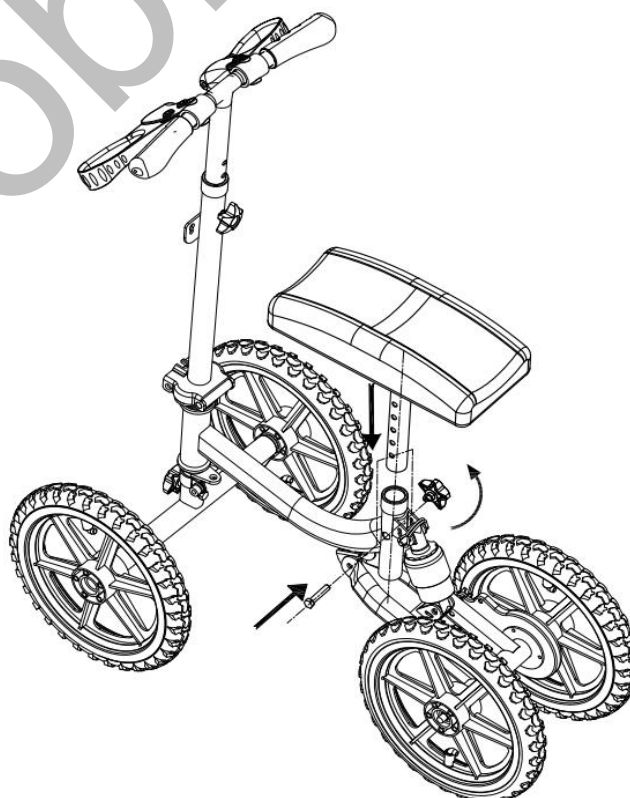STEP 7: To adjust the brakes

Use an Allen key to loosen the hand lever, reposition it and pull it into the desired position. The brake tension can be adjusted using the nuts at both ends of the brake cable. Turn clockwise to tighten and anti-clockwise to loosen. Activate the parking brake by simultaneously pulling the brake lever and pressing the small silver locking button. Release the brake lever (with the locking button pressed) and the brake remains in an applied position; the Mobi-Roll should be fixed. The brake is released by pulling the brake lever again; the locking button then pops out audibly.

If the Mobi-Roll can still be moved despite the parking brake being applied, you can re-tension both the brake levers and the lower part of the drum brake. This is necessary after some use - similar to a bicycle brake.

Operating instructions

To operate the Mobi-Roll, place your hands on the handlebars.

Place the knee of your injured leg comfortably on the centre of the knee platform.


Release the brake to move slowly forwards or backwards, take small steps with your other leg and keep your injured leg centred on the knee platform.

When you move the handlebars to the left or right, the steering axle is turned with minimal effort.

Turns should only be performed at a slow movement speed.

Product

Parameters

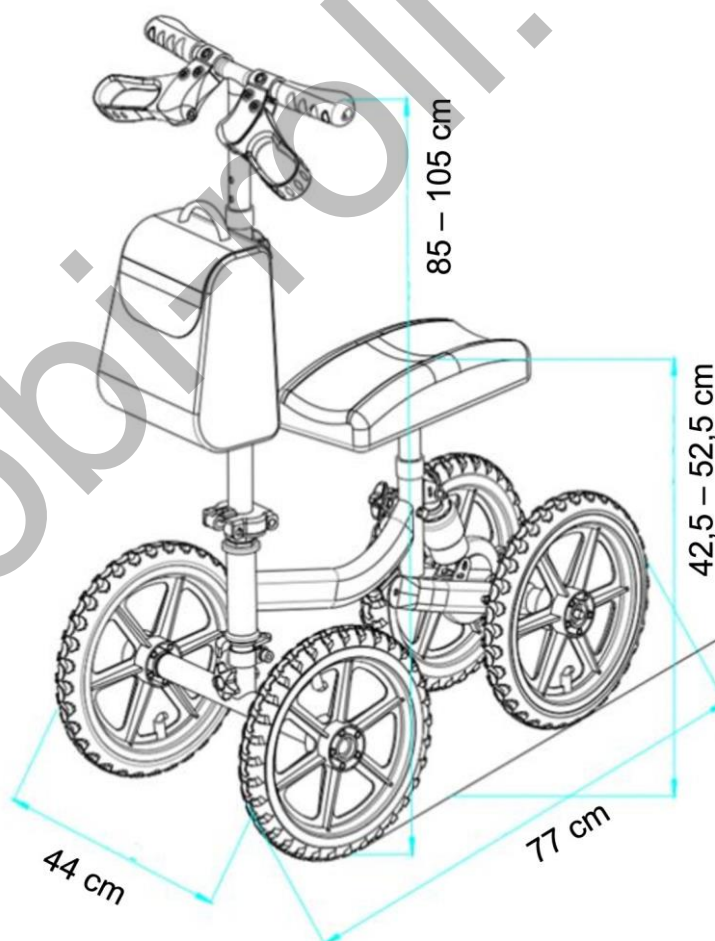The Mobi-Roll CC model has undergone the following changes:

Length 82 cm

Front axle width 44 cm

Height of knee support; 47.5 - 57.5 cm

Handlebar height: 90-110 cm

Air pressure: 2.0 - 3.0 bar, depending on application, can be topped up with any vehicle air pressure device (petrol stations)

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